


Хирургический протокол Taper ULTRA KIT


Хирургический протокол Taper Ultra KIT TSIII (Длина: 10 мм)

Ø6.0mm

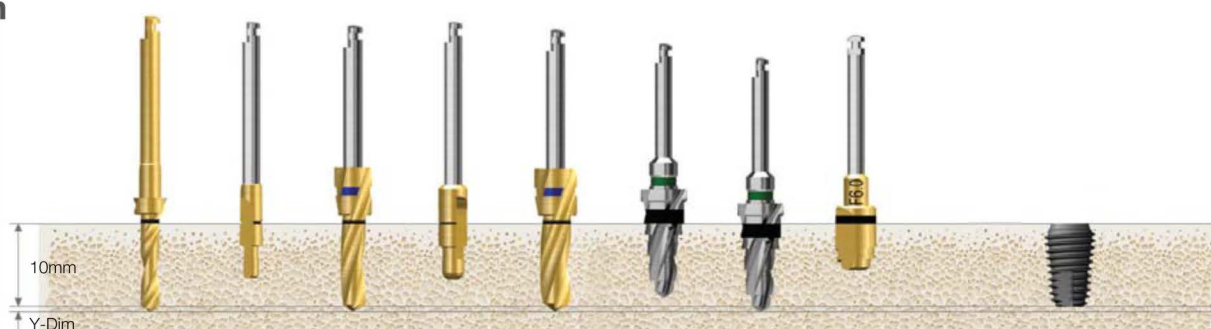

Качество кости	Ø 2.2 Drill	Ø 2.0/3.0 Pilot Drill	Ø 3.0 Drill	Ø 3.0/3.8 Pilot Drill	Ø 3.8 Drill	F6.0 Taper Drill	F6.0 Taper Drill	F6.0 Cortical Drill	Ø 6.0 Fixture
Мягкая	▶	▶	▶	▶		▶			Установка имплантата
Нормальная	▶	▶	▶	▶	▶		▶		
Твердая	▶	▶	▶	▶	▶		▶	▶	

Ø7.0mm

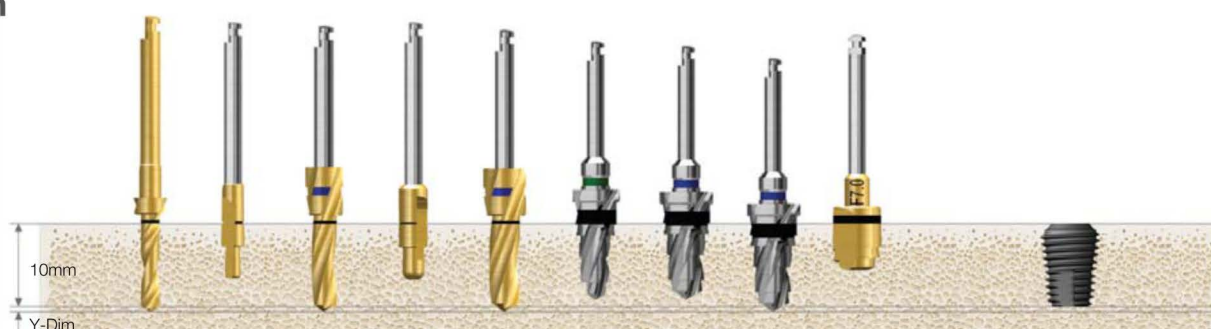


Качество кости	Ø 2.2 Drill	Ø 2.0/3.0 Pilot Drill	Ø 3.0 Drill	Ø 3.0/3.8 Pilot Drill	Ø 3.8 Drill	F6.0 Taper Drill	F7.0 Taper Drill	F7.0 Taper Drill	F7.0 Cortical Drill	Ø 7.0 Fixture
Мягкая	▶	▶	▶	▶		▶	▶			Установка имплантата
Нормальная	▶	▶	▶	▶	▶	▶		▶		
Твердая	▶	▶	▶	▶	▶	▶		▶	▶	

Хирургический протокол Taper ULTRA KIT


Хирургический протокол Taper Ultra KIT TS VI (Длина: 10 мм)

Ø6.0mm

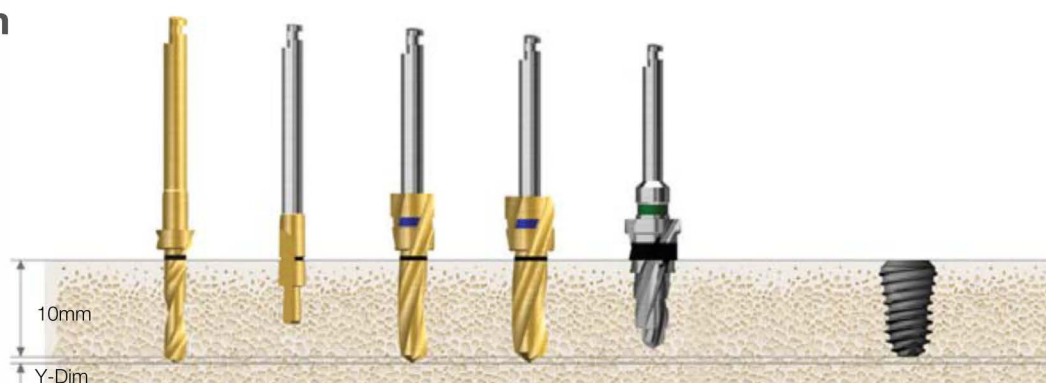

Качество кости	Ø 2.2 Drill	Ø 2.0/3.0 Pilot Drill	Ø 3.0 Drill	Ø 3.8 Drill	F6.0 Taper Drill	Ø 6.0 Fixture
D4	▶			▶		Установка имплантата
Мягкая	▶	▶	▶		▶	

Ø7.0mm

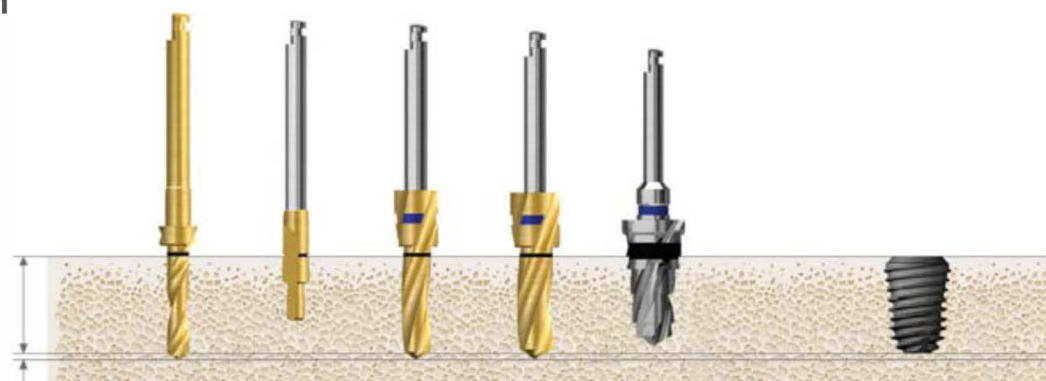

Качество кости	Ø 2.2 Drill	Ø 2.0/3.0 Pilot Drill	Ø 3.0 Drill	Ø 3.8 Drill	F7.0 Taper Drill	Ø 7.0 Fixture
D4	▶			▶		Установка имплантата
Мягкая	▶	▶	▶		▶	